AudioNova Reusable Earplugs

EN User Guide
BG Ръководство за потребителя
CS Návod k použití
DA Brugsværne
DE Gebrauchsanweisung
EL Εργαλείο χρήσης
ES Instrucciones para el usuario
ET Kasutusjuhend
FI Käyttöohjeet
FR Mode d'emploi
HR Upute za uporabu
HU Felhasználói útmutató
IT Istruzioni per l'utente
LT Naudojimo instrukcija
LV Liešanas pamācība
NL Gebruikershandleiding
NO Bruksanvisning
PL Instrukcja obsługi
PT Instruções de utilização
RO Instrucțiuni de utilizare
RU Руководство пользователя
SK pokyny na používanie
SL Uporabna navodila
SV Användningsinstruktorer

EN | English

USER INSTRUCTION
THE PRODUCT CONSISTS OF TWO PARTS; THE PILL SHAPED FILTER AND THE REUSABLE UNIVERSAL FILTER (Image A) (excludes SleepTight). The product consists of two parts only and is not a medical device.

Drive - AA125
AUDIONOVA DRIVE REUSABLE EARPLUGS
SNR 24 dB H 23 dB M 22 dB L 21 dB NRR 16 dB
CE certification data

F [Hz]	63	125	250	500	1000	2000	4000	8000
MA (dB)	23.9	23.2	22.3	22.7	24.8	30.3	22.5	36.7
SD (dB)	3.4	3.0	2.6	2.6	3.6	3.3	2.9	3.5
APV (dB)	20.5	20.2	19.7	20.1	21.2	27.5	19.6	33.2
ANSI S3.19 / CSA Z94.2 test results								
F [Hz]	125	250	500	1000	2000	3150	4000	8000
MA (dB)	22.1	20.7	20.5	23.1	31.1	21.8	22.7	33.7
SD (dB)	3.0	2.8	3.6	3.8	4.0	5.1	3.2	3.4
APV (dB)	16.1	15.1	13.3	16.7	23.1	-	18.4	-
AS/NZS 1270:2002								
F [Hz]	125	250	500	1000	2000	4000	8000	
MA (dB)	22.7	21.4	19.3	24.0	29.4	28.7	24.0	
SD (dB)	4.5	3.7	4.8	4.0	3.0	3.4	4.4	
MA-SD (dB)	18.2	17.7	14.5	20.0	25.4	25.4	19.6	
SLC(0) = 22dB Class 4								

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1. Inserting into the ear
 1.1 Ensure the filter is correctly positioned in the ear tip (Image G, H & I).

1.2 Grip the ear tip between thumb and forefinger (Image B).

1.3 Pull the top of the ear outward and upward to ease insertion: This is best achieved by holding the ear with the opposite hand or the arm placed around the back of the head (Image C).

1.4 Position the tongue at the top of the ear making it easier to find and grip when removing the tip from the ear (Image D).

1.5 Gently twist and twist the product into the ear until it sits comfortably in the ear canal whilst forming a good seal with the ear canal.

2. Removing, Cleaning and Storage
 1. Find and grip the tongue of the ear tip (Image D).
 2. Remove the product with a slow twisting motion to gradually break the seal with the ear canal. This also releases any tension that may be felt during removal.
 3. Clean the ear tip after each use to remove earwax (Cerumen) and debris. The ear tips can be wiped clean using a damp cloth or cotton bud. After cleaning remove the product in a clean reusable protective case or bag. Do not store with other objects. Recommended storage temperature range is between 10-20°C.

After removing the filter (3. Changing ear tip size), the ear tips can be cleaned with soap or hygiene tablets and then rinsed in tepid water and dried thoroughly.

3. Changing ear tip size
 1. Remove the filter from the current ear tip: Gently squeeze the ear tip just below the filter until 'it pops' out of the ear tip (Image E).

2. Insert the new filter into the ear tip: Gently grip the filter as you turn the filter to align it with the ear tip (Image F).

3. Press down on the logo to correctly position the filter in the cavity of the ear tip (Image G, H & I).

4. Insert the ear tip into the ear canal by following the instructions (1. Inserting into the ear).

WARNING
 1. ENSURE THE FILTER IS CORRECTLY POSITIONED IN THE EAR TIP (Image G, H & I).

2. Grip the ear tip between thumb and forefinger (Image B).

3. Pull the top of the ear outward and upward to ease insertion: This is best achieved by holding the ear with the opposite hand or the arm placed around the back of the head (Image C).

4. Position the tongue at the top of the ear making it easier to find and grip when removing the tip from the ear (Image D).

5. Gently twist and twist the product into the ear until it sits comfortably in the ear canal whilst forming a good seal with the ear canal.

6. Removing, Cleaning and Storage
 1. Find and grip the tongue of the ear tip (Image D).
 2. Remove the product with a slow twisting motion to gradually break the seal with the ear canal. This also releases any tension that may be felt during removal.
 3. Clean the ear tip after each use to remove earwax (Cerumen) and debris. The ear tips can be wiped clean using a damp cloth or cotton bud. After cleaning remove the product in a clean reusable protective case or bag. Do not store with other objects. Recommended storage temperature range is between 10-20°C.

After removing the filter (3. Changing ear tip size), the ear tips can be cleaned with soap or hygiene tablets and then rinsed in tepid water and dried thoroughly.

7. Changing ear tip size
 1. Remove the filter from the current ear tip: Gently squeeze the ear tip just below the filter until 'it pops' out of the ear tip (Image E).

2. Insert the new filter into the ear tip: Gently grip the filter as you turn the filter to align it with the ear tip (Image F).

3. Press down on the logo to correctly position the filter in the cavity of the ear tip (Image G, H & I).

4. Insert the ear tip into the ear canal by following the instructions (1. Inserting into the ear).

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