Cleaning Tablets

gebruiksaanwijzing
gebrauchsanweisung
user instructions
instrukcja użycia

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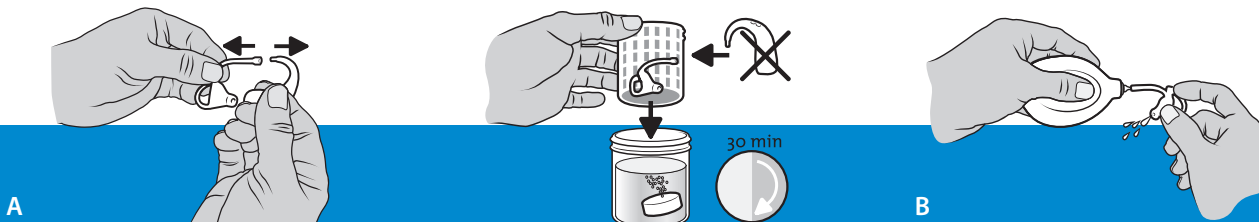
EN Cleaning Tablets for hearing aids

USE: Fill $\frac{3}{4}$ of the container with lukewarm water and add a cleaning tablet. Disconnect the ear mould from the hearing aid and place it in the container. Leave to soak for at least 30 minutes (A). Rinse the ear mould with clean water and proceed to the next step. Use the airball to air dry water in the earpiece and the tube (B). The tab can be used only once. **CONTAINS:** 40,5% sodium bicarbonate, 19% potassium peroxymonosulfate, 16% citric acid, 14% sodium carbonate, 9% polyethylene glycol, sodium metaphosphate, peppermint oil, sodium benzoate, indigo. Store at room temperature / store in a dry place / do not expose to direct sunlight.

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PRECAUTIONS: Keep out of reach of children. Read label before use. Causes serious eye damage. Contains Peppermint oil: May cause an allergic reaction. If medical advice is needed, have product container or label at hand. **IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a doctor/physician.

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