HUMANTECHNIK

*lisa*

**DE**

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Seite 2

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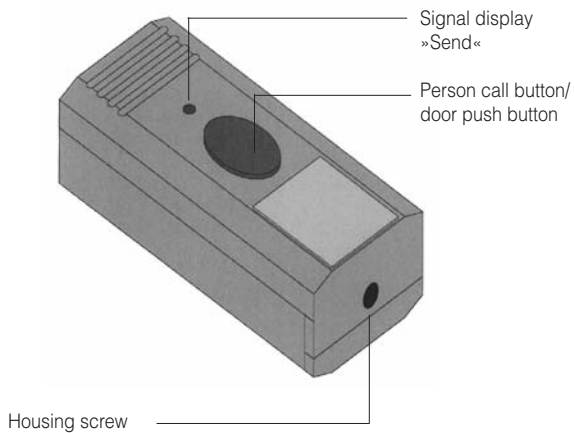
Congratulations on purchasing your *RF person call button/RF door push button*. We hope it will give you many years of reassuring operation. Please read through these operating instructions carefully. They tell you how to set up the system correctly and explain all the system's features to you.

### Included in the standard package:

Please check if all following components are included:

- *lisa RF person call button/RF door push button*
- 1.5 V battery (type: lady)
- Operating Instructions
- Warranty card

If any parts are missing please immediately contact your dealer or the manufacturer directly.

### How it works

A signal unit consists of at least one transmitter and one receiver. The *RF person call button/RF door push button* transfers signals wire-free (radio signal 868.35 MHz) to the receiver.

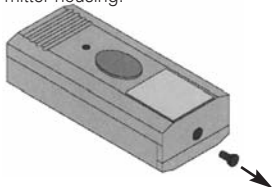
### Special features

A recognised signal is transmitted for a duration of 15 seconds. This increases reliability and range.


## Initial operation

### 1. Inserting the battery


1. Release the screw on the transmitter housing.

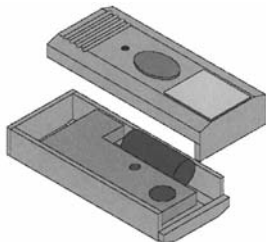2. Remove the top of the housing.

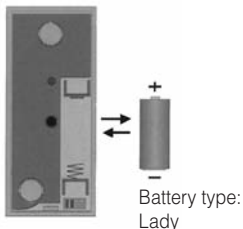
3. Insert a new battery (ensure polarity is correct!)

## 2. Setting the correct radio channel

Thanks to a special digital transmission process the radio signals can be smoothly recognised. This procedure enables 10 different radio channels to be set. The default setting on all units is channel 0. The units only need to be set to different radio channels if units are being operated by several users within the same building.

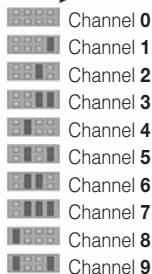Open the housing as described under the "Inserting the battery" section. You can set the radio channel desired using three small short circuit bridges.

## Please note

All transmitters and receivers must be set to the same channel.

## Triggering the person call signal/door bell signal

By pressing the person call button/door push button a signal is triggered on each receiver set to the same channel as the transmitter.

## Radio range

The transmitter is designed to have a radio signal range of 80 meters under optimum conditions.

The following conditions may result in a reduced range:

- Signal transmission through building structures or vegetation
- Interference from TVs, computer monitors and mobile phones.

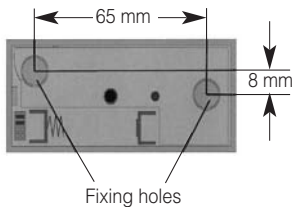
Under certain circumstances, this may prevent the receiver from functioning at all.

- Positioning of the transmitter at an unfavourable distance from surfaces which reflect radio waves, for example floors and walls. This may lead to the signal wave being weakened or even cancelled out by the reflected wave.

- Metallic objects reduce the range due to the shielding effect of the metal.
- Particularly in towns and cities, there may be many other sources of radio waves which can disrupt the original signal.
- Having units operating at similar frequencies located close to one another may also result in mutual interference.

### Wall assembly (only for RF door push button)

Please use the 5 mm wall plugs included with the *RF door push button*. Use a masonry drill bit to drill the holes required at the position you desire (see diagram below).

### Battery life

Under normal conditions (7 transmissions a day), the transmitter battery should last for about 1 year. To check the level of the battery press the transmitter's person call button. If the "Send" signal display does not illuminate the battery has to be changed!

### Maintenance and care

The *RF person call button/RF door push button* does not require any maintenance. If the unit does become dirty, simply wipe it clean with a soft, damp cloth. Never use spirits, thinners or other organic solvents.

Do not set up the *RF person call button/RF door push button* where it will be exposed to full sunlight for long periods. In addition, it must be protected against excessive heat, moisture and severe mechanical shocks.

**Note:** This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

Please make sure that the batteries are not exposed to excessive heat from such as sunlight, fire or anything similar.

## Warranty

The *RF person call button/RF door push button* unit is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly. This warranty covers the repair of the product and returning it to you free of charge. It is recommended that you send in the product in its original packaging, so keep the packaging for the duration of the warranty period.

The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit). Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt.

**Always specify the product number in any event.**

**Disposal** of used electric and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system).

The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electric and electronic units. You protect the environment and health of your fellow men by the correct disposal of this products. Environment and health are endangered by a faulty disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of this product from your local community, your communal disposal company or your local dealer.

## Technical data

Power supply:	1,5 V-battery (type: Lady)
Battery service life:	approx. 1 year
Frequency:	868.35 MHz
Maximum transmitting power:	<10 mW
Operating range:	0° C – 40° C
Height:	24 mm
Width:	84 mm
Depth:	26 mm
Weight:	48 g

This device satisfies the following EC directives:

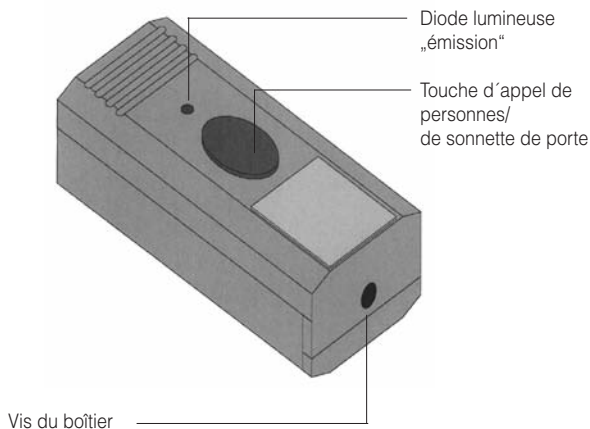
2011 / 65 / EC RoHS-directive

2012 / 19 / EC WEEE-directive

2014 / 53 / EC RED-directive

Compliance with the directives listed above is confirmed by the CE seal on the device. CE compliance declarations are available on the Internet at [www.humanteknik.com](http://www.humanteknik.com).

Technical specifications subject to change without prior notice.

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**Sarabec Ltd**