# Hygiene kit for hearing aids

gebruiksaanwijzing  
gebrauchsanweisung  
user instructions  
instrukcja użycia  
mode d'emploi  
istruzioni d'uso  
használati útmutató  
brugsanvisning

<b>NL</b>	<b>GEBRUIKSAANWIJZING</b> .....	<b>3</b>
<b>DE</b>	<b>GEBRAUCHSANWEISUNG</b> .....	<b>6</b>
<b>EN</b>	<b>USER INSTRUCTIONS</b> .....	<b>9</b>
<b>PL</b>	<b>INSTRUKCJA UŻYCIA</b> .....	<b>12</b>
<b>FR</b>	<b>MODE D'EMPLOI</b> .....	<b>15</b>
<b>IT</b>	<b>ISTRUZIONI D'USO</b> .....	<b>18</b>
<b>HU</b>	<b>HASZNÁLATI ÚTMUTATÓ</b> .....	<b>21</b>
<b>DK</b>	<b>BRUGSANVISNING</b> .....	<b>24</b>

## Reinigingspray

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
## Cleaning Spray

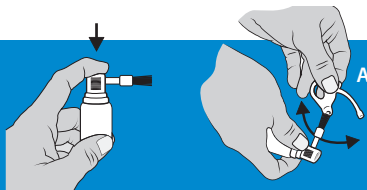**USE:** Cleans your hearing aid / ear mould. Gently push until the brush is moist. Carefully brush the openings of the earpiece (A). Leave to dry for about 3 minutes.

**CONTAINS:** 5% sodium lauryl ether sulfate, 1% lemon oil, 1% carrageenan, methylisothiazolinone

**PRECAUTIONS:** Keep out of reach of children.

Read label before use. If skin irritation or rash occurs: get medical advice/attention. May cause an allergic skin reaction. If medical advice is needed, have product container or label at hand. Avoid breathing spray. Store at room temperature / store in a dry place / do not expose to direct sunlight.


## Cleaning Wipes

**USE:** First thoroughly clean the surface with a cleaning wipe; allow the liquid to work for one minute before drying it with a dry (paper) towel. One cleaning wipe is sufficient for a surface of about 30x30 cm. The wipes can only be used once.

**CONTAINS:** 1% lemon oil, cocamidopropyl betaine, 2-propanol, 2-phenoxyethanol

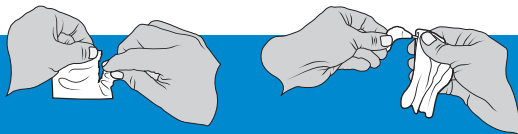**PRECAUTIONS:** Keep out of reach of children. If skin irritation or rash occurs: Get medical advice/attention. May cause an allergic reaction. If medical advice is needed, have product container or label at hand. Store at room temperature / store in a dry place / do not expose to direct sunlight.


EN

## Cleaning Tablets

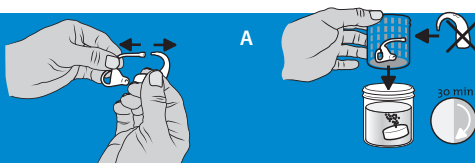**USE:** Fill  $\frac{3}{4}$  of the container with lukewarm water and add a cleaning tablet. Disconnect the ear mould from the hearing aid and place it in the container. Leave to soak for at least 30 minutes (A). Rinse the ear mould with clean water and proceed to the next step. The tab can be used only once.

**CONTAINS:** 40,5% sodium bicarbonate, 19% potassium peroxymonosulfate, 16% citric acid, 14% sodium carbonate, 9% polyethylene glycol, sodium metaphosphate, peppermint oil, sodium benzoate, indigo.


**PRECAUTIONS:** Keep out of reach of children. Read label before use. Causes serious eye damage. Contains Peppermint oil: May cause an allergic reaction. If medical advice is needed, have product container or label at hand. **IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a doctor/physician. Store at room temperature / store in a dry place / do not expose to direct sunlight.

## Airball

(B) Use the airball to air dry water in the earpiece and the tube.

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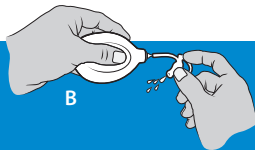
## Drying Capsules

**USE:** Place the unopened tablet in the cup with the cardboard side up (A). Switch off the hearing aid and open the battery compartment, place in the cup and close it (B). Leave overnight (C). The tablet can be used for about one month and has to be replaced when the silicon granules have turned white.

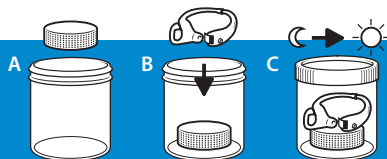**CONTAINS:** 98% silica gel


**PRECAUTIONS:** Keep out of reach of children. Read label before use.

Store at room temperature / store in a dry place / do not expose to direct sunlight.

## Comfort Gel

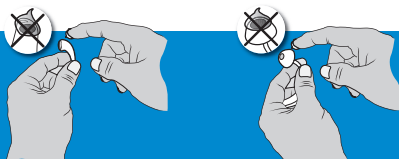**USE:** The gel makes inserting and removing your hearing aid or ear mould easier and ensures a pleasant feeling. Apply the gel to the surface inserted into the ear canal. Ensure no gel enters the openings to prevent them from getting blocked.

**CONTAINS:** 5% sodium lauryl ether sulfate, 5% cocamide monoethanolamine, 1% coco dimethylamine oxide, 1% lemon oil, 1% carrageenan, methylisothiazolinone - Store at room temperature / store in a dry place / do not expose to direct sunlight.

**PRECAUTIONS:** Keep out of reach of children. Read label before use. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Causes serious eye irritation. If medical advice is needed, have product container or label at hand.

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WAKE UP INTERNATIONAL B.V.